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**CE** 0123 - DIR 93/42/EEC

## Symbols on product and packages

|  |   |
|--|---|
|  | Sterilization using irradiation   |
|  | Not for reuse in intended applications as defined by the manufacturer                   |
|   | Use by  |
|  | Caution, general warning symbol<br>Caution, see documentation supplied with the product |
|  | Date of manufacture   |

## Intended use

The instruments are used for the cutting, dissection, grasping, and coagulation of tissue in minimally invasive surgery.

## Safe handling and preparation

DANGER

**Infection hazard for patients and/or users and impairment of product functionality due to reuse. Risk of injury, illness or death due to contamination and/or impaired functionality of the product!**

- ▶ Do not reprocess the product.

CAUTION

**Risk of breakage in the jaw area! The instrument contains fine ceramic components for insulation purposes.**

- ▶ To avoid damage to the working tip, especially to the ceramic insulation: Do not apply excessive forces and protect the working tip from impacts and knocks.

The product is gamma-sterilized and supplied in sterile packaging.

The product must not be reused.

- ▶ Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- ▶ Read, follow, and keep the instructions for use.
- ▶ Use the product only in accordance with its intended use, see Intended use.
- ▶ Store any new or unused products in a dry, clean, and safe place.
- ▶ Prior to each use, inspect the product for loose, bent, broken, cracked, worn, or fractured components.
- ▶ Do not use the product if it is damaged or defective. Set aside the product if it is damaged.
- ▶ Do not use products that are in open or damaged sterile packaging.
- ▶ Do not use the product after its use-by date.

WARNING

**Risk of injury from ignition or explosion of flammable gases! Sparks may occur when using the HF device as directed.**

- ▶ Observe the safety guidelines in the instructions for use of the HF device.

WARNING

**Thermal injuries to patients/users due to insufficient insulation of leads in active accessories!**

- ▶ Adjust the HF device to an appropriate setting to ensure that the peak output voltage does match or not exceed the accessory voltage rating specified for the product.

- ▶ Adjust the HF power output to the intended surgical intervention. Take into account clinical experience or reference values.
- ▶ Select the lowest possible HF power output.
- ▶ Keep the product's contact surfaces clean during surgery. Remove encrusted tissue residues or body fluids with a moistened swab.

The product is furnished with the following connection on the plug side: Aesculap flat connector

Refer to our brochures to find a compatible cable.

The accessory voltage rating of the product is 700 Vp.

The accessory voltage rating must exceed or match the peak output voltage with which the product is operated in combination with a suitable HF device at an appropriate operating mode/setting (see IEC/DIN EN 60601-2-2).

To avoid HF burns:

- ▶ Always keep the working end of the product in the user's field of vision whenever the HF power is activated.
- ▶ Prior to activating the HF device, check that the working end of the product is not touching any electrically conductive accessories.
- ▶ Prior to each use, visually inspect the product for: damage or surface changes to the insulation.
- ▶ Never place the product on or next to the patient.
- ▶ When using accessories for endoscopy or laparoscopy, deactivate the automatic switch-on mode of the HF device.
- ▶ Follow the instructions for use of the HF device.

## Safe operation

### Bipolar coagulation

From the health-and-safety perspective, bipolar coagulation is the superior alternative to monopolar coagulation, because the current does not flow through the whole body, but only through a narrowly defined tissue region between the jaw pieces/scissors blades of the instrument.

#### Note

If both jaw pieces of the grasping forceps come into direct contact, coagulation cannot occur as the power source is short-circuited.

- ▶ Make certain the grasping surfaces are clean and of a polished-metal appearance.
- ▶ Grasp and compress the tissue in such a way that it is in contact with both metal jaw pieces.
- ▶ Switch on the HF current.

### Bipolar cutting

- ▶ Compress the tissue between the two scissor blades.
- ▶ Switch on the HF current.
- ▶ Cut through the tissue mechanically. The slower the cut, the larger the coagulation zone at the edge of the cut.

### Storage

- ▶ Store sterile packed single-use products dust-protected in a dry, dark and temperature-controlled room.

## Technical Service

### Service addresses

Aesculap Technischer Service  
Am Aesculap-Platz  
78532 Tuttlingen / Germany  
Phone: +49 (7461) 95-1602  
Fax: +49 (7461) 16-5621  
E-Mail: ats@aesculap.de

Other service addresses can be obtained from the address indicated above.

## Accessories/Spare parts

For accessories and spare parts, see brochure C304, C094 and C766 or visit <http://www.endoscopy-catalog.com>

## Disposal

- ▶ Adhere to national regulations when disposing of or recycling the product, its components and its packaging!

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