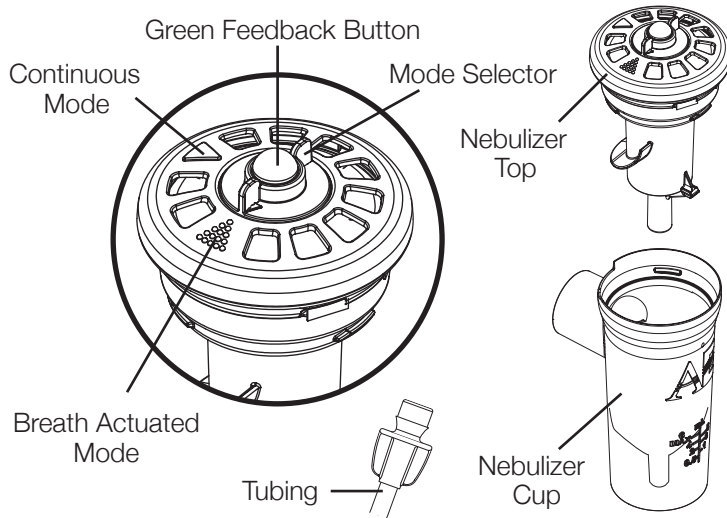
AeroEclipse™ II

BAN™ Nebulizer without mouthpiece

REF 105508

Trudell Medical International
725 Baransway Drive, London, Ontario,
Canada N5V 5G4
+1-519-455-4862
info@trudellmed.com
www.trudellmed.com

**Trudell Healthcare Solutions /
Trudell Solutions Santé**
758 Baransway Drive, London, Ontario,
Canada N5V 5J7
customerservice@trudellhs.com
www.trudellhs.com

Instructions for Use

1. Prior to use, inhale and exhale through the mouthpiece to verify the movement of the green feedback button located on top of the device.
2. Unscrew and remove top of nebulizer. Place prescribed medication into nebulizer cup (maximum cup volume 6 mL). Reattach top and gently hand tighten.
3. Attach one end of the supplied tubing to the fitting in the bottom of the nebulizer and the other end to the flowmeter or compressor. Ensure both ends are securely engaged.
4. Set the flowmeter to 7 to 8 litres per minute (Lpm) with an air source capable of delivering 50 psi (344.7 kPa).
5. Place mouthpiece in mouth, and inhale slowly and deeply. As you inhale, the green feedback button on top of the nebulizer will move into the fully down position, indicating that the **AeroEclipse® II BAN™** Nebulizer is producing aerosol in response to inhalation. Exhale normally.
6. Continue to inhale from the nebulizer until nebulizer begins to sputter, or as instructed by your physician or qualified healthcare worker.

The **AeroEclipse® II BAN™** Nebulizer can be operated in Continuous Mode by rotating the mode selector 90 degrees clockwise to the Continuous Mode position (△). The nebulizer can be returned to Breath Actuated Mode by rotating the mode selector 90 degrees counterclockwise to the breath actuated position (⋯).

Cleaning Instructions

Rotate the mode selector to the Breath Actuated position (green feedback button is up, mode selector has rotated counterclockwise) for both the washing and drying cycles. Between treatments, remove the supplied tubing from the bottom of the nebulizer, unscrew and remove top of nebulizer and wash nebulizer parts in a warm water and liquid dish detergent solution. Rinse in clean warm water and allow parts to dry thoroughly before reassembling.

Single Patient Use

The device should be replaced after approximately 7 days or immediately if components are cracked or the nebulizer does not function properly. This device is not intended to be reprocessed and should not be autoclaved or boiled.

Indications for Use

The **AeroEclipse® II BAN™** Nebulizer is intended to be used by patients who are under the care or treatment of a licensed healthcare provider. The device is intended to be used by these patients to administer aerosolized medication prescribed by a physician or healthcare provider. The intended environments for use include the home, hospitals and clinics.

WARNING

Close supervision is necessary when this product is used by or near infants, children or anyone requiring assistance to use the device. It contains parts that may present a choking or strangulation hazard. **DO NOT LEAVE CHILDREN UNATTENDED DURING TREATMENTS.**

CAUTIONS

- The **AeroEclipse® II BAN™** Nebulizer can deliver aerosolized medication in either a Breath Actuated or Continuous Mode. In Breath Actuated Mode, medication is delivered on inspiration only and may result in an increase of the dosage to the lungs. Healthcare providers should take this information into consideration with regards to dosage.
- Failure to properly follow the above cleaning instructions may have an adverse affect on device performance.

REORDER NUMBER

AeroEclipse® II BAN™
Nebulizer without mouthpiece

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