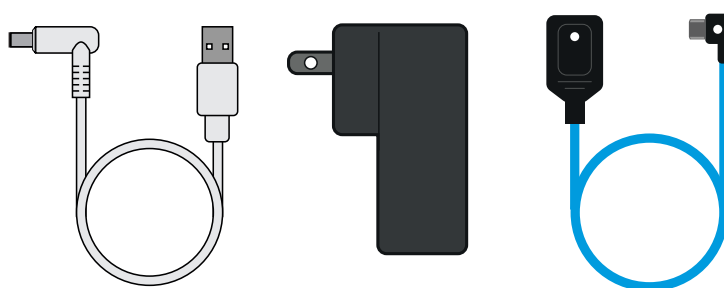
PROVU VIDEO STYLET 8" DISPLAY

CS	DISPLEJ PROVU VIDEO STYLET 8"
NL	PROVU VIDEO STYLET 8" DISPLAY
FR	MONITEUR POUR STYLET VIDÉO PROVU 20 CM (8 POUCES)
DE	PROVU VIDEO STYLET 8-ZOLL-DISPLAY
IT	DISPLAY PROVU VIDEO STYLET 8"
RO	AFIŞAJ PROVU VIDEO STYLET 8"
ES	MONITOR DE 8 PULGADAS PARA VIDEOESTILETE PROVU
SV	PROVU VIDEOMANDRÅNG 8" DISPLAY
TR	PROVU VİDEO TARZI 8" EKRAN

EN PROVU VIDEO STYLET 8" DISPLAY

Technical Specifications

DISPLAY	
RESOLUTION (PIXELS)	800 X 1280
ORIENTATION	Landscape
DISPLAY TYPE	8"
START UP TIME	Immediate
SOFTWARE VERSION	Refer to device screen while charging
CONNECTIONS	
USB CONNECTION	Micro USB
POWER CONNECTION	DC Power
STORAGE AND TRANSPORTATION	
TEMPERATURE	10°C TO 40°C (50°F TO 104°F)
RELATIVE HUMIDITY	≤85%, non-condensing
ATMOSPHERIC PRESSURE	700 to 1060 hPa
OPERATING CONDITIONS	
TEMPERATURE	20°C +/-10°C
RELATIVE HUMIDITY	≤85%, non-condensing
ATMOSPHERIC PRESSURE	700 to 1060 hPa
DISPLAY ELECTRICAL POWER RATING	
POWER INPUT	Input: 5.0V --- 1.5A
BATTERY TYPE	3.7V, 5000 mAh, Lithium-ion
TYPICAL BATTERY RUN TIME	≈ 7 hours (of continuous use)
TYPICAL BATTERY CHARGE TIME	≈ 7 hours (from zero charge)
WALL PLUG ELECTRICAL POWER RATING	
INPUT	100-240V AC 0.5A 50-60Hz
OUTPUT	Max 5.2V DC --- 3.0A
DIMENSIONS	
WIDTH X HEIGHT X THICKNESS	205mm x 130mm x 25mm
WEIGHT	396g
PLUG	
PLUG TYPE	110-120V with NEMA 5-15P 3 Pin Plug

Intended Use

The ProVu Video Stylet (VS) 8" Display is a digital display intended to show live images from the ProVu Video Stylet (ProVu VS) only, via the micro USB cable, as an aid to intubation of adults in hospital environments.

Product Description

The ProVu VS 8" Display is a non-sterile, reusable digital display intended to display live images from the ProVu VS. When connected to the ProVu VS, a live video is transmitted from the device, which can be used to further aid intubation of the patient.

The Display can be charged with the provided ProVu charging cable, in between uses or while in use. When fully charged, the ProVu VS 8" Display will provide up to 7 hours of continuous use. The product can also be charged through the micro USB port if the provided DC charger is unavailable.

Flexicare does not recommend the modification of the device. Any attempts to do so will result in the failure of the device.

Precautions

- Ensure there is adequate room for easy operation.
- Inspect the ProVu VS 8" Display and all accessories for damage prior to each use.
- Charge the Display fully before use. For shipping requirements, ProVu VS 8" Displays are shipped with zero charge.
- Before charging, ensure that all components are completely dry, including the micro USB port.
- If using the Display Pole Stand, slide the Display securely onto the connector of the articulating arm, ensuring it is fully seated.
- If using the ProVu Display Table Stand, slide the Display securely onto the connector, ensuring it is fully seated.
- Always place the Display Table Stand on a stable, flat surface.

Warnings & Cautions

- ProVu VS 8" Display and Accessories should only be used by trained personnel.
- Do not use within Magnetic Resonance Imaging (MR/MRI) environments or with other high interference sources.
- Do not sterilise the device.
- The use of devices and other accessories which are not approved by Flexicare may result in sub-optimal performance and place the patient at risk.
- The ProVu VS 8" Display is designed to fit on either the Display Pole Stand or the Display Table Stand. Use of another device may result in damage and/or loss of function to the ProVu VS 8" Display.
- The ProVu VS 8" Display should only be used with the ProVu VS and ProVu accessories.
- Always disconnect the ProVu VS 8" Display from the charging cable before cleaning.
- Ensure the voltage recommendations of the Display are compatible with power supplies.
- Do not clean the Display or Display accessories using any abrasive agents or mineral acids.
- Do not use sterilants with caustic ingredients, such as surgical scrub solutions, peroxide solutions, bleaches, or povidone-iodine solutions.
- Do not disassemble device.
- Inadequate battery levels may result in sudden failure without warning. Check sufficient battery level before use.
- Do not modify the device.
- Device is not servicable.
- The battery is not removable. Do not open or remove battery. Warranty void if opened.

Any serious incident that occurs in relation to the device should be reported to the manufacturer and the competent authority of the Member State of the user and/or patient.

Setting Up the ProVu VS Stands

Refer to instructions supplied with the ProVu VS Pole Stand (IP0000-030) or ProVu Display Table Stand (IP0000-021).

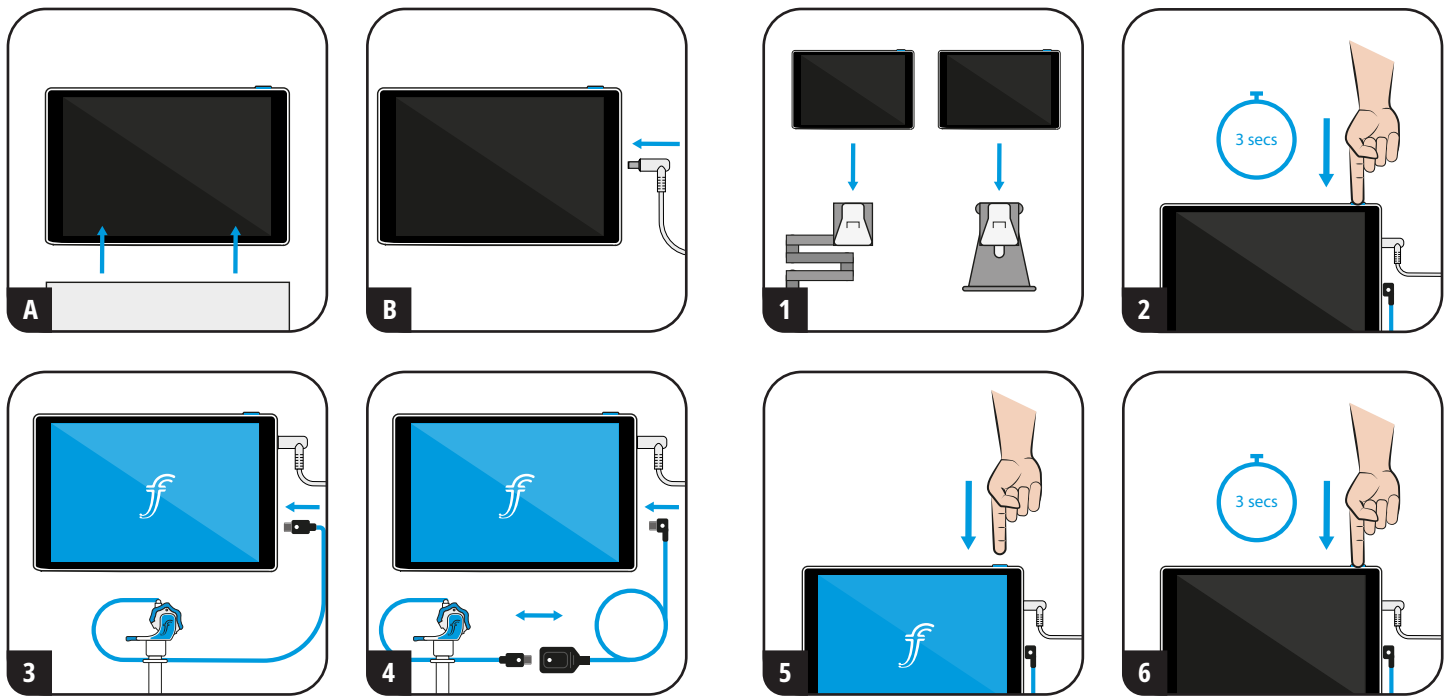
Charging

1. Connect the ProVu VS 8" Display to the DC charging cable, connected either to the plug socket on the Display Pole Stand or directly to an outlet.
2. When connected and turned on, an indication of charge is shown on the screen. If this screen does not display, check the security of the connections.
3. When fully charged, the Display will provide up to 7 hours of continuous use.
4. For optimal performance, keep the Display connected to the charger while in use. If this is not suitable, re-charge between uses or when prompted on screen.

Using with ProVu VS

For more detailed instructions on the use of ProVu VS, refer to ProVu VS instructions (IP0000-042) supplied with the ProVu VS.

1. Secure the ProVu VS 8" Display either on the ProVu VS Pole Stand or the Display Table Stand, according to user preference.
2. Press and hold the Power Button for 3 seconds.
 - If charge capacity is more than 5%, a blue screen with a connection prompt image should appear.
 - If charge capacity is 5% or less, a blue screen with the battery icon with strike out will appear. Please connect to charger; the Display can be charged while simultaneously in use with a ProVu VS.
3. Connect the ProVu VS to the ProVu VS 8" Display using the micro USB connector and observe the Display screen to verify that a live image is being received from the ProVu VS.
4. If using the reusable Amplifier Cable, connect the elbow of the Amplifier Cable to the Display and insert the ProVu VS camera line to the connector of the Amplifier Cable. To ensure this is done correctly, line up the dots on the cables as shown in image #4.
5. To put the ProVu VS 8" Display into standby mode, press the Power Button once quickly and the screen will turn black. To wake up from standby mode, press the Power Button once quickly and the screen image will resume.
6. To turn off the ProVu VS 8" Display, press and hold down the Power Button for 3 seconds and the screen will turn black.


ProVu VS Software

The ProVu VS 8" Display has several on-screen prompts to assist use.


Software Version

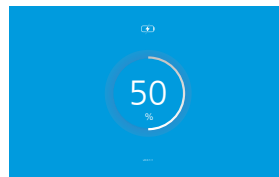Software version is displayed in the bottom of the display when connected to power.


Display is Charging

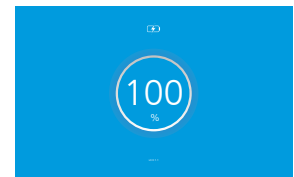ProVu VS 8" Display battery is charging. Percentage charge is indicated in the circle.


Charged

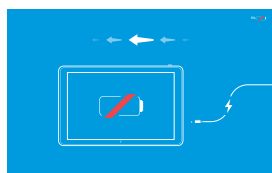ProVu VS 8" Display is fully charged.


Charge Display

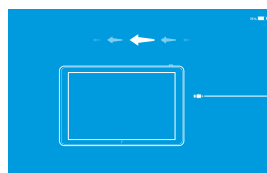Display has insufficient battery. Please charge via USB or charging dock.


Connect to ProVu VS

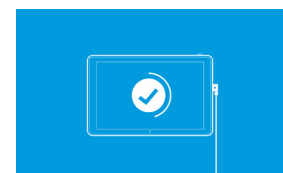Connect ProVu VS 8" Display to view camera image.

Connected to ProVu VS

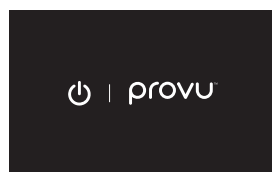ProVu VS 8" Display is connected and camera image is loading.


Turn Off

If Power Button is held for 3 seconds or more, this screen will show while the display shuts down.

Error 1

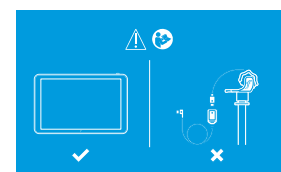Software error or crash. Reset ProVu VS 8" Display unit by holding Power Button for 5 seconds or more.




Error 2

Camera error or crash. Replace ProVu VS and/or Amplifier Cable.

ProVu VS Battery Indicator

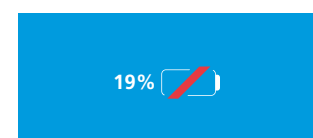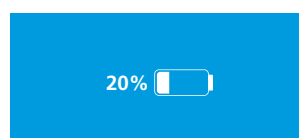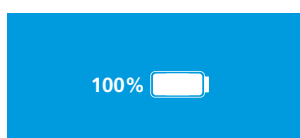During use, the battery indicator appears on screen to show battery power remaining. The icon will change to flash on and off when battery charge is less than 20%.

Troubleshooting

PROBLEM	
No live image is seen	
POSSIBLE EXPLANATION	ACTION NEEDED
ProVu VS 8" Display needs to be charged	Charge display via the DC Charger or micro USB cable. Note that the display can be charged while in use.
ProVu VS 8" Display may be in sleep mode	Press the Power Button once to wake.
ProVu VS 8" Display is damaged	Replace the ProVu VS 8" Display with a new device.
ProVu VS 8" Display may be turned off	Press and hold the Power Button for 3 seconds.
ProVu VS Amplifier Cable is damaged	Replace the ProVu VS Amplifier Cable with a new device.
Incorrect connection to ProVu VS 8" Display and/or Amplifier Cable	Remove and reconnect the ProVu VS 8" Display to the device via the micro USB port as shown in the instructions for use section.
ProVu VS may be damaged	Replace with a new ProVu VS device.
PROBLEM	
Low image quality	
POSSIBLE EXPLANATION	ACTION NEEDED
Incorrect connection to ProVu VS 8" Display and/or Amplifier Cable	Remove and reconnect the ProVu VS 8" Display to the device via the micro USB port as shown in the instructions for use section.
ProVu VS Amplifier Cable is damaged	Replace the ProVu VS Amplifier Cable with a new cable.
Dirty screen	Wipe with an IPA cloth.
Damp screen	Wipe with a dry cloth.
Camera lens may have secretions on it	Clean camera lens with clean Microfibre towel or replace ProVu VS.
Illumination failure	Replace with a new ProVu VS.
Incorrectly inserted cable	Ensure all connections are secure.

List of Accessories

- Cables & Mounting Accessories
- Charging Cable
- 2m Amplifier Cable
- ProVu VS Pole Stand
- ProVu Display Table Stand

After Use

1. Disconnect the ProVu VS 8" Display from the power supply.
2. Disconnect the ProVu VS from the ProVu VS 8" Display, or from the Amplifier Cable if being used.
3. After use, dispose of all single-use components in accordance with local policies and / or as advised by your local healthcare provider.
4. Clean and dry the ProVu VS 8" Display and Amplifier Cable (if used) referring to the cleaning instructions.

Cleaning

1. Wear gloves throughout the cleaning procedure.
2. Clean the ProVu VS 8" Display and reusable accessories after each use, using a germicidal wipe, e.g. 70% isopropyl alcohol wipe.
3. Inspect the ProVu VS 8" Display and accessories for contamination. If any is present repeat steps 1 & 2.
4. Once the ProVu VS 8" Display and accessories are free from contaminants, dry with a clean, dry cloth.
5. Check device for damage. If device does not function, refer to troubleshooting steps.

Warranty Statement

Flexicare Medical Limited warrants that the product purchased meets the labelled specifications of the product and will be free from defects in materials and workmanship that occur within one (1) year from the date of purchase.

This warranty does not cover damage caused by:

- Handling during shipping.
- Use or maintenance contrary to labelling and instructions, including but not limited to incorrect disassembly, reprocessing and reassembly.
- Alteration or repair by anyone not authorised by Flexicare Medical Limited.

Abuse, Misuse or Accidental Damage

If a product covered by this warranty is determined to be defective materials, components, or workmanship, and the warrant claim is made within the warranty period described above, Flexicare Medical Limited will, at its discretion, repair or replace the defective product free of charge.

Written authorisation must be obtained from Flexicare Medical Limited to return the product prior to sending it with postage paid to Flexicare Medical Limited's designated agent.

This warranty is in lieu of all other warranties, expressed or implied, including but not limited to, the implied warranties of merchantability and fitness for a particular purpose. Flexicare Medical Limited's obligation under this warranty is limited to repair or replacement of products containing a defect.

Flexicare Medical Limited is not responsible for any indirect or consequential damages resulting from a product defect covered by the warranty.

This warranty shall be governed by and construed in accordance with Flexicare Medical Limited's Terms and Conditions.

Symbols Glossary

Catalog Number

Keep Dry

Serial Number

Recycle

Product Conforms with Directive 93/42/EEC

Separate Collection

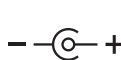
Refer to Instruction Manual/Booklet

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