

# How to Open the **DUAL TOP** Bottle Effectively



## **TWIST OFF!**

1. Hold the bottle firmly with thumb and forefinger at the **bottle NECK**.
2. With the other hand hold the **TWIST tab** with thumb and forefinger. Pull up and twist the tab a full 180° until the tab comes off.

## **SNAP OFF!**

1. To snap off the top for a larger opening, with one hand hold the bottle firmly in the **MIDDLE of the bottle**.
2. With the other hand pull back the **bottle NECK** along the vertical line until it snaps off.



For more information call your Territory Manager!  
Customer Service **1.800.265.5494**



**TRUDELL**  
HEALTHCARE SOLUTIONS

Visit us at [www.trudellhs.com](http://www.trudellhs.com)



# How to Open the TWIST tab Bottle Effectively



## TWIST OFF!

1. Hold the bottle firmly.
2. With the other hand hold the **TWIST tab** with thumb and forefinger. Pull up and twist the tab a full 180° until the tab comes off.



**TRU TEST**®

A Tradition of Trust.

For more information call  
your Territory Manager!

Customer Service **1.800.265.5494**

Visit us at [www.trudellhs.com](http://www.trudellhs.com)



**TRUDELL**  
HEALTHCARE SOLUTIONS